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Latex free

INSTRUCTIONS FOR THE PATIENT

Progressive Therapeutic Vaginal Dilators for:

- Vaginismus and Dyspareunia
- Vaginal reconstructive surgery
- After radiation treatments

C.N. 337448.4

Warning: please read these instructions before starting treatment. The instructions are provided solely for information purposes. It is recommended to follow the instructions of the specialist in order to use these progressive therapeutic vaginal dilators.

Manufactured by: Eurogine, s.l.
Antoni Gaudí, 91
08830 Sant Boi de Llobregat, Barcelona
www.eurogine.com

Ref.: 11260700 - U-rev.: 201005

donea is a registered trademark of Eurogine s.l.

INDICATIONS

Donea progressive therapeutic vaginal dilators are indicated to treat dysfunctions as Vaginismus and Dyspareunia, and to open and give elasticity to the vagina in cases of reconstructive surgery or after radiation treatments.

MAINTENANCE

This set of progressive therapeutic vaginal dilators has been designed using a material that is comfortable to the touch, giving it slight flexibility so that insertion and removal are easily achievable by the patient.

- After each session, carefully wash the Donea dilators with warm water and mild soap.
- Dry them thoroughly before storing them back in their case until their next use.
- Keep them away from heat sources and in a dry place.

DISINFECTION / DECONTAMINATION

Rinse thoroughly with water before re-using the dilators after disinfection with cold liquid solutions.

CONTRAINDICATIONS

- Do not use if you are pregnant or suspect being pregnant.
- Do not use in case of vaginal infection.
- Consult with your specialist.

GENERAL INSTRUCTIONS

Your therapist has recommended that you use this set of donea progressive therapeutic vaginal dilators. As indicated by your therapist, begin treatment with the smallest diameter dilator. Once inserted into the vagina, you will feel mild pressure. Gradually, during the period indicated by your specialist, you must use the dilators with increasing

diameter until you feel entirely comfortable. Whenever you start a workout session, do so with a dilator whose diameter already fits comfortably and poses no difficulties.

We recommend creating a suitable environment where the patient is alone, unhurried, comfortable and relaxed, to start treatment. After lying on your back with legs bent, open, and relaxed, you can begin your workout.

If needed, use a water-base lubricant to facilitate insertion.

Check with your specialist if you have any questions.

BEFORE YOU BEGIN YOUR WORKOUT

Practice relaxing the muscles around the vagina, known as the pubococcygeal muscles (PC), following the steps listed below:

- Squeeze your PC muscles as if trying to stop urinating and count to three.
- Relax.
- Repeat steps 1 and 2 ten times.
- Perform this workout known as Kegel exercises, 2 times daily.

CONDUCTING THE WORKOUT STEP BY STEP

Step 1

- Start with the smallest dilator.
- Using a mirror will allow you to see the genital area.
- Apply a small amount of lubricant to insert the dilator. DO NOT use vaseline, baby oil, or other oil-based lubricants because they are not soluble in water and can be irritating to the vaginal walls.
- Tense your muscles as you practiced, and relax them before inserting the dilator into the vagina. Each time you contract the PC muscle, hold the dilator so that it does not slip out. As soon as it is relaxed, push the dilator a little farther into the vagina.

- NEVER force the introduction of the dilator. When you are relaxed, it will slide and you will notice some mild pressure

Step 2

- Insert and hold the dilator in the vagina for several minutes. Repeat until you are able to keep the dilator for ten minutes without discomfort.
- You may have to hold the dilator in place with one hand to keep it from slipping out.
- Make several attempts over several days of working out.

Step 3

- When you are able to perform the insertion comfortably and without feeling any discomfort, move them gently from side to side. Slide it in and out.
- Make several attempts over several days of working out, until no discomfort is felt.

When successfully completing step 3, you are ready to start again with step 1 using the next size dilator.

Contact your doctor whenever you deem appropriate.

MEASURES

Set of six progressive therapeutic vaginal dilators donea		
Size	Length. mm.	Diam. mm.
Extra-small	125	10,50
Small	130	16,00
Medium	135	21,50
Large	140	27,00
Extra-large	150	32,50
Extra extra-large	160	38,00

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