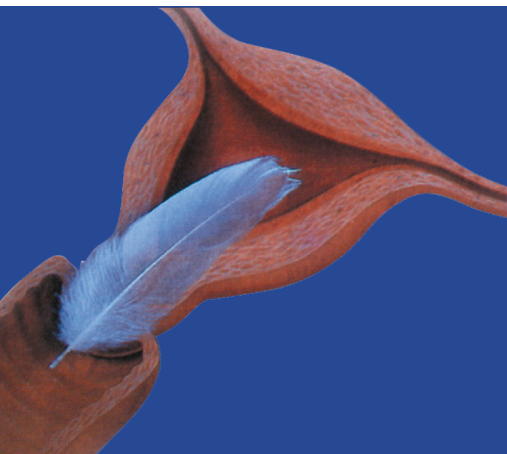
contracepción-ginecología

euro  gine

Technology for Women's Healthcare

35
YEARS
ANNIVERSARY

Dispositivos intrauterinos
anticonceptivos con cobre
Copper-bearing intrauterine
contraceptive devices

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Copper-bearing intrauterine contraceptive device

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EUROGINE is a company with more than 30 years in the health sector, dedicated to the research, development, manufacture and distribution of copper-bearing intrauterine devices for contraception for the domestic and international market.

EUROGINE manufactures and supplies its copper IUDs not only with the conviction that they will satisfy the demands of HCPs, who today know Eurogine devices, and which are prescribed with total satisfaction, but also to make available a wide range of world-renowned IUDs as a reliable hormone-free alternative for contraception that satisfies all women who prefer a long-acting contraceptive with no side-effects related to hormones.

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What is a copper-bearing IUD?

It is a small flexible plastic device with a copper wire wrapped around its vertical shaft that is inserted into the uterus through the vagina and cervical canal during a gynaecologic appointment to prevent pregnancy.

Some types do not have a plastic frame. Some others also have copper sleeves on the horizontal arms. But all of them have a knotted thread on the vertical shaft that hangs from the cervix towards the vagina to control its position and presence, as well as for its removal.

The copper IUD is an ideal contraceptive method for most women of childbearing age, regardless of whether they have had children, and can be used until menopause is established.

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Copper is a metal that occurs naturally throughout the environment, in rocks, soil, water, and air. Copper is an essential element in plants and animals (including humans), which means it is necessary for us to live.

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The copper IUD uses one single active and natural ingredient (copper) to prevent pregnancy, and it is a long-lasting contraceptive method. It can also be used as emergency contraception if it is implanted within 5 days after unprotected intercourse.

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The mechanism of action of copper IUDs is local and is mainly produced by copper, which is a natural spermicide, so it limits the motility of the spermatozoa, thus preventing it from fertilizing the egg. Copper IUDs also cause a normal inflammatory reaction in the uterine mucosa that may prevent implantation.

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How effective is a copper IUD?

The copper IUD is a long-lasting contraceptive method, the effectiveness of which is very high. It is established that less than 1 pregnancy occurs for every 100 women during the first year of use. A small risk of pregnancy remains after the first year and continues as long as the copper IUD is in use.

The copper IUD can be used as emergency contraception if it is implanted within 5 days after unprotected intercourse.

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The copper IUD is safe and suitable for nearly all women

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Who can use a copper IUD?

Most women can use copper IUDs safely and effectively, including women who:

- Have or have not had children
- Are adolescents or women over 40 years old
- Have just had an abortion or miscarriage if there is no evidence of infection
- Are breastfeeding
- Do hard physical work
- Have had an ectopic pregnancy
- Have had pelvic inflammatory disease (PID)
- Have vaginal infections
- Have HIV clinical disease that is mild or with no symptoms

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Contraceptive benefits

Copper IUDs have been determined by the WHO as the best option for birth control due to its excellent cost-effectiveness-innocuousness relationship. They are safe and suitable for nearly all women.

Their contraceptive benefits are very high in comparison to their disadvantages, which can be considerably reduced with a correct choice of shape and/or size, and with an accurate implantation.

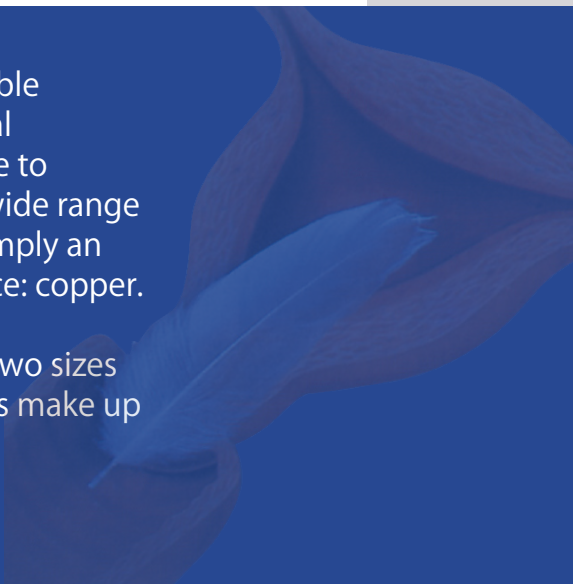
Copper IUDs have a very low rate of risks and complications. Still, these can happen, and additional medical treatment may be needed.

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Eurogine makes a reliable alternative to hormonal contraception available to women by offering a wide range of copper IUDs with simply an active natural substance: copper.

Two different shapes, two sizes and 3 insertion systems make up Eurogine IUDs profile.

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- They are a contraceptive method of long duration and low rate of complications
- Women return to previous fertility immediately after removal
- Do not interfere with sexual intercourse, ovulation, or breastfeeding
- Do not contain hormones and do not require rest periods
- Do not cause side effects typical of medicalized devices (hormones)
- May help protect against endometrial or cervical cancer
- Reduce the risk of ectopic pregnancy
- No daily or nightly routines are required. It is enough with a simple monthly auto-check and periodic check-ups with the healthcare provider

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Copper-bearing intrauterine contraceptive devices

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EUROGINE, based in a very important background on this field, had the goal of supplying to the market a modern range of copper IUDs.

The company started its expansion worldwide in 1998 towards important and demanding markets such as Germany and Sweden. Nowadays EUROGINE exports to about 50 countries worldwide. This success is due to the continuous improvement of its products, based on very important investments on R&D+i to keep always the highest level of quality, diversity and technology.

All these efforts took EUROGINE to develop the GOLD T® IUD in 1998, which became a referent in the field of non-hormonal contraception, and which is now exported to most of the countries in Europe and the Middle East, and also in Latin America.

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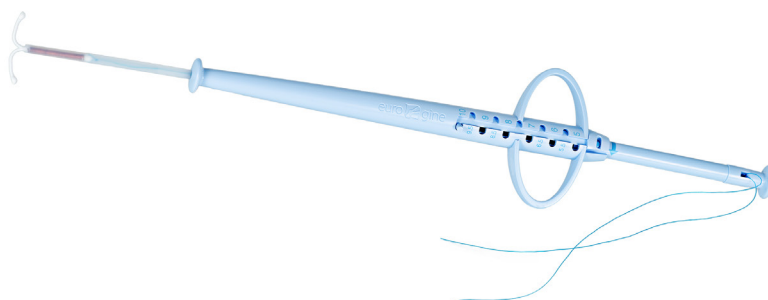
 Gold T®

The Gold T® IUD is the only IUD that has a copper wire with a gold core.

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 Gold T®

The incorporation of the gold core, like the silver core in other IUD models, provides greater resistance to fragmentation for the copper wire.

In addition, the GOLD T® IUD offers an adjustable insertion system that provides a high degree of precision in the placement of the IUD and, by means of a safety piston, reduces the risk of perforation.

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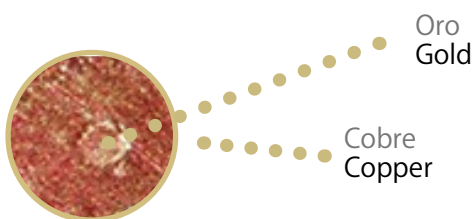
 Gold T[®]

 Gold T[®]

- 99.99% pure copper wire with a 0.1 mm diameter gold core that provides greater resistance to fragmentation of the copper wire
- 375 mm² of active copper surface
- Peculiar "T"-shaped frame (Υ) that facilitates the insertion and removal procedures
- 2 sizes available (Normal and Mini) which permit to fit different uterus sizes
- Adjustable insertion system that provides a high degree of precision in the placement of the IUD
- Safety piston that reduces the risk of perforation
- Maximum time "in situ": 5 years

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The “Y”-shaped frame eases a smooth and gentle insertion and removal of the device due to the horizontal arms bending upwards in both procedures with the consequent minimizing of the diameter that needs to pass through the cervical canal either in the insertion or the removal.

EUROGINE® T 380 IUD is available in two variants depending on the wire having or not having a silver core.

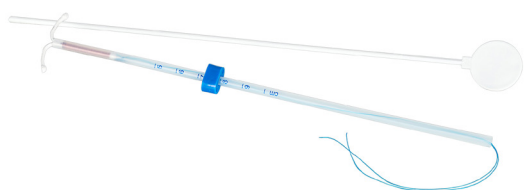
Both EUROGINE® T 380 Ag and EUROGINE® T 380 Cu offer 2 sizes to fit almost all uterus.

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eurogine® T 380

Y-shaped IUDs to ease their insertion and removal

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eurogine® T 380

- 99.99% pure copper wire or 99.99% copper wire with a 0.1 mm diameter silver core that provides greater resistance to fragmentation for the copper wire
- 380 mm² of active copper surface
- Peculiar “Y”-shaped frame that facilitates the insertion and removal procedures
- 2 sizes available (Normal and Mini) which permit to fit different uterus sizes
- “Pull and Push” insertion system: the IUD must be preloaded into the insertion tube in order to be placed into the uterus
- Maximum time “in situ”: 5 years

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The “Ω”-shaped frame of ANCORA 375 Cu IUD allows a one hand insertion into the uterus by pushing the IUD directly through the cervical canal without the horizontal arms having been preloaded in the insertion tube beforehand.

- 99.99% pure copper wire
- 375 mm² of active copper surface
- “Ω”-shaped frame
- “Push” insertion system: by pushing the IUD through the cervical canal
- Maximum time “in situ”: 5 years

Ref.	Descripción Description	Ancho Width	Largo Length	Núcleo Ag Silver Core
01030000	Áncora 375 Cu	20 mm	35 mm	

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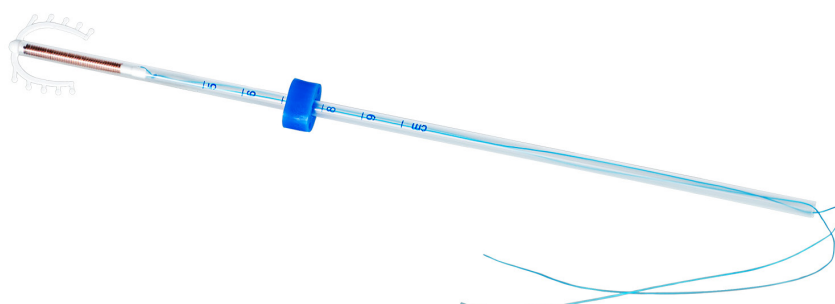
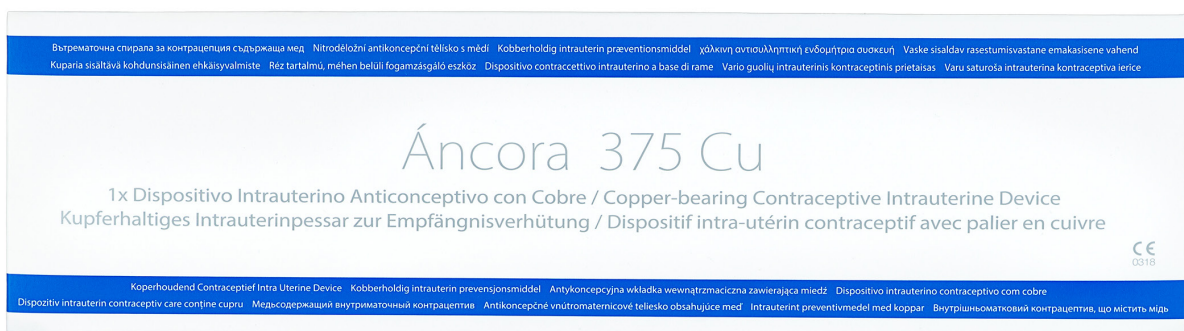
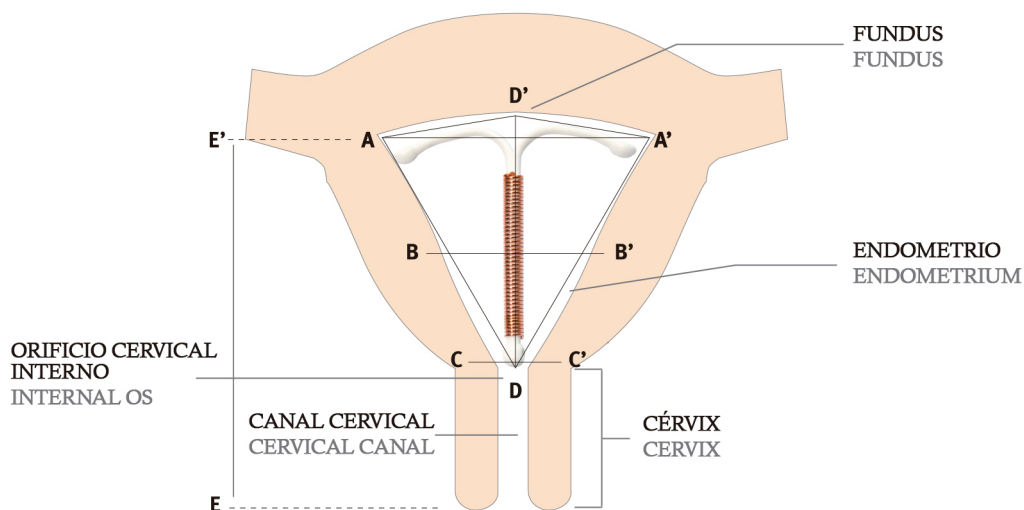
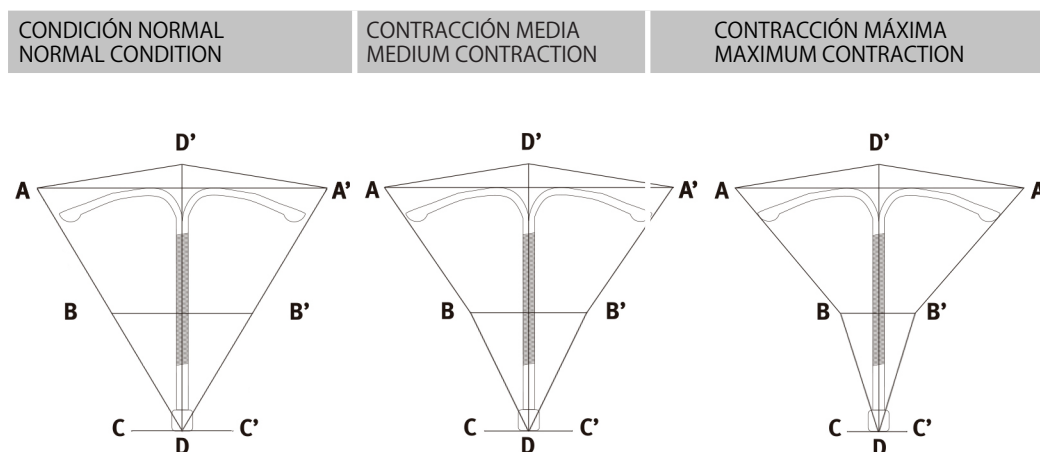
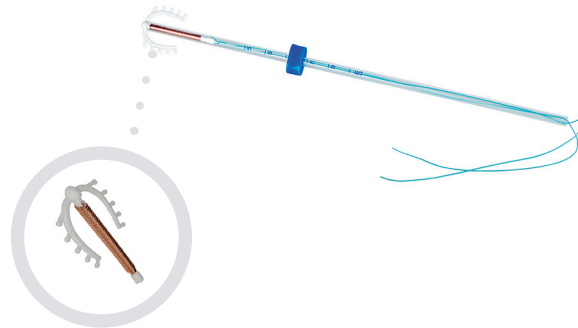
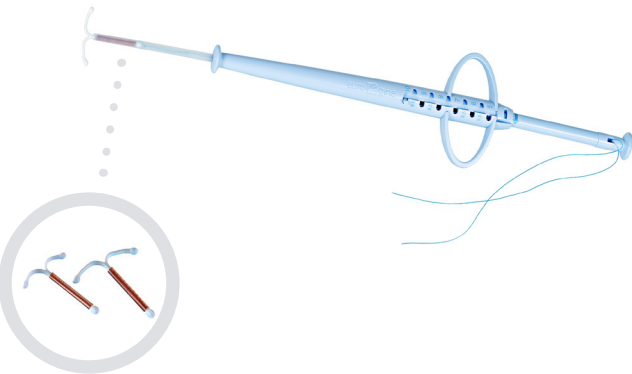
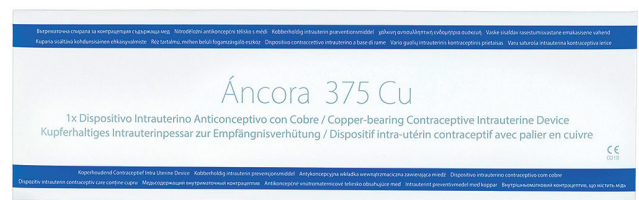
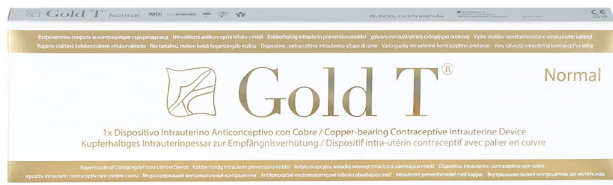
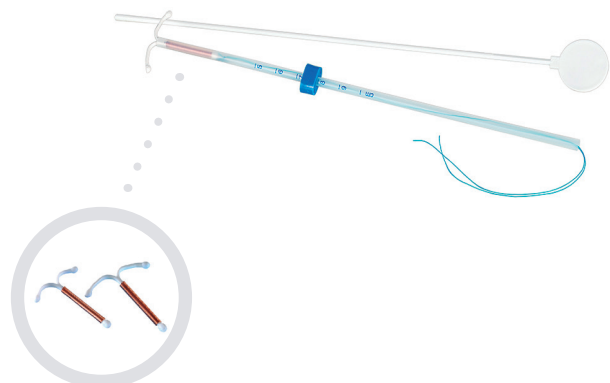
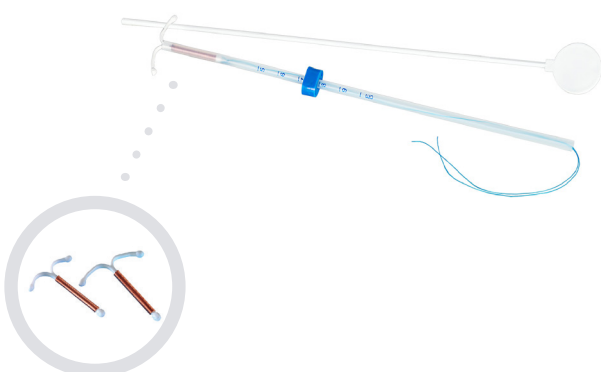
Eurogine worldwide

Spain, Germany, France, Italy, Austria, Portugal, United Kingdom, Netherlands, Luxembourg, Belgium, Greece, Switzerland, Sweden, Finland, Denmark, Norway, Russia, Latvia, Estonia, Ukraine, Czech Republic, Slovakia, Hungary, Bulgaria, Poland, Romania, Moldova, Albania, Serbia, Bosnia & Herzegovina, Montenegro, North Macedonia, Kosovo, Malta, Cyprus, Turkey, UAE, Jordan, Lebanon, Oman, Bahrain, Saudi Arabia, Kuwait, Qatar, Iran, Iraq, Egypt, Libya, Morocco, Israel, Colombia, Ecuador, Mongolia, Vietnam, Kazakhstan, Uzbekistan.

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